
Single Use Hydrophilic Urinary Catheter With Water Sachet Instructions for use

hi-slip® plus hydrophilic urinary catheters are intended for intermittent emptying of the urinary bladder.

All hi-slip®, hi-slip® plus hydrophilic catheters are for single use only!

Reuse will deteriorate the hydrophilic surface and may lead to urethral damage and/or urinary tract infections!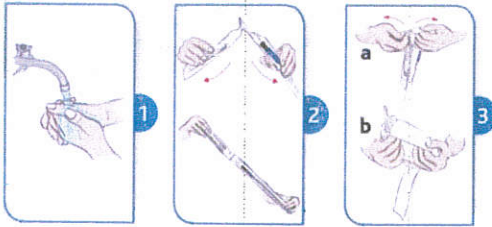

1. Wash your hands before catheterization.

2. Open the packaging fully out.

3.(a) Open the water sachet package (almost half of the package) by peeling the tabs without touching the water sachet. (b) Hold the water sachet by folding film and paper side.

4.(a) Open the catheter package 1-2 cm by peeling the tabs on connector side. (b) Place the tip of the water sachet into the catheter packaging.

5.(a) Burst the water sachet by squeezing.

(b) You can stick the package onto any dry surface using the self-adhesive tape on the package. Make sure that the catheter is immersed in water for at least 15 seconds. During this time prepare for catheterization.

6. By using the blue handling strip to hold on to, you can perform catheterization easily without ever touching the catheter. If needed, handling strip can be removed before or during catheterization.

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